



VALUE OF ADAMTS-13 IN CASES OF TROMBOTIC MICROAUGIOPATHIES SEINTAQUIDAGONOIM DITORMORT 40 SESAD NI ST-STMADA 40 SISYLANA

3175, Côte Sainte-Catherine Road, Montreal, QC H3T 1C5 (514) 345-4931 ext. 5681

		Patient Informatio	2	
	equesting Institution/Unit:		"	
A	ddress: Civic number Street	Last Name First N	ame	
	Province/Country			
	Postal code:	Gender: F	М	
Р	Phone number: FAX:			
		Healt care Instituti		
K	equesting Physician :		Health facility file #	
s	ampling Date: Y/M/D Time_			
s	ampled By:	STAT Or Stamp the patient	's Health Care Institution card	
		Stamp the patient	3 realth care institution card	
CLINICAL INFORMATION FORM TO ATTACH TO SAMPLES				
CLINICAL INFORMATION FORM TO ATTACH TO SAMPLES				
SUSPECTED PATHOLOGY:				
» Thrombotic thrombocytopenic purpura (TTP) :				
» Hemolytic-uremic syndrome (HUS) : Typical Atypical				
	PRIOR EPISODES of thrombotic microangiopathy: NO YES Specify dates: Y/M/D			
ACCOCIATED BATHOLOGIES .				
	ASSOCIATED PATHOLOGIES: Pregnancy Weeks preg Cancer Infection Transplant			
	Medication Specify: Other	er Specify:		
	CLINICAL CONTEXT THAT CORRESPONDS TO AN ACUTE STACE.			
	CLINICAL CONTEXT THAT CORRESPONDS TO AN ACUTE STAGE: ** Fever: NO YES			
	» Neurological signs : NO YES Specify :			
	» Abdominal signs : NO YES Specify :			
PIOLOCICAL DADAMETERS.				
BIOLOGICAL PARAMETERS : » Hemolytic anemia : NO YES				
	» Thrombopenia : NO YES			
TREATMENT: Time of collection in relation to treatment:				
Time of concedion in reliation to declarient.				
If treatment, which one?:				
SAMPLING PROCEDURES AND SHIPMENT OF BLOOD SAMPLES				
	SAMPLING PROCEDUR	ES AND SHIPMENT OF BLO	JOD SAMPLES	
>	Collect peripheral venous blood samples using 4,5 ml citrate tubes containing 3,2% (0,109 mM) sodium citrate solution.			
	Except for specific cases, collect blood before treatment, such as plasma transfusion and / or plasma exchange.			
	Note: Do not collect samples using EDTA or HEPARIN.			
>	Centrifuge the blood samples at 2 500g or more for			
			On the package ➤ Write the laboratory's complete address	
	Decant plasma into several aliquots (minimum of 3) of at least 500μL each		Write the name of the originating hospital	
>	Freeze the aliquots of plasma at -20°C or lower until they are ready to be shipped on dry ice		and laboratory (you)	
			➤ Indicate on the package how many samples it contains	
All of the aliquots must be clearly identified (Patient's family name, given name, birth date, sample date and to			> Insert the requisition forms (F-726) in a	
	(ratient's family name, given name, birth date, s	sample date and time)	sealed Ziploc bag and place them in a	
>	The aliquots must be shipped along with the clin	ical information form (F-726)	separate compartment from the samples	
>	Ship the aliquots on dry ice to the laboratory: CHU Sainte-Justine			
	omp the anquots on ary lee to the laboratory.	Hemostasis Laboratory, 2 nd floor, Unit 6,	Room 2610, C/O Anik Cormier	
	Information: (514) 345-4931 #7170	3175, Côte Sainte-Catherine Road Montreal, QC, Canada H3T 1C5		